Fig. 1

Vehicle Lighter Outlet or Power Outlet
Sortie de courant ou sortie
d'allume-cigare de voiture
Sigarettenaansteker- of stroomcontact
Bilens lighteruttak eller strømuttak
Tomada de Isqueiro para Veículo ou
Tomada Eléctrica
Uttag för cigarettändare eller strömuttag

#90 Plug Assembly
Assemblage de fiche n° 90
Nr. 90 steekcontact
Nr. 90 pluggmontering
#90 Ficha
Nr 90 proppenhet


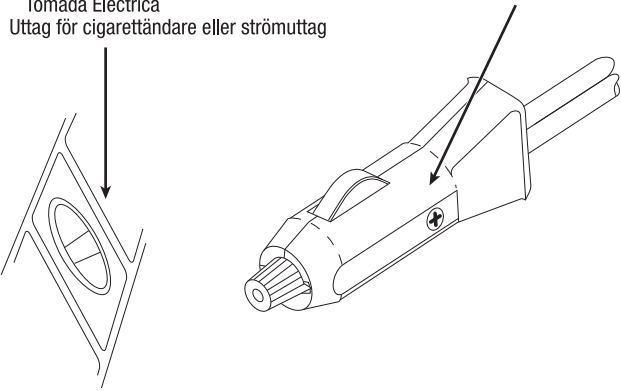
Fig. 2

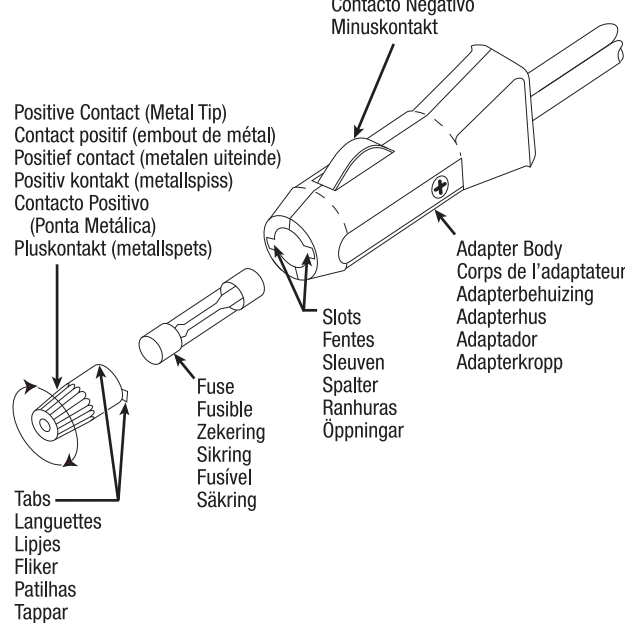NL Autoadapter

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US Car Adapter

For use with the Purely Yours/Lactaline Electric Breast Pump

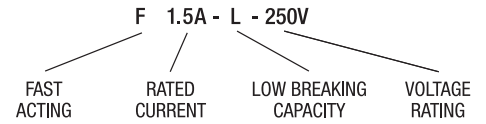
Connect the car adapter to the power receptacle located on the left side of the pump. Plug the other end into the vehicle lighter receptacle of your vehicle.

Connecting the breast pump with a car adapter must only be made with the Ameda Car Adapter (Fig. 1).

Fuse Replacement Instructions (Fig. 2)

1. Push in metal tip at the end of the adapter. Rotate counterclockwise until tabs line up with slots in the adapter body.
2. Pull out metal tip and fuse.
3. Visually inspect fuse element, if blown, replace fuse.
4. Insert fuse into the adapter. Line up tabs of metal top with slots in adapter body, push in and rotate clockwise to lock.

Replacement Fuse Characteristics:

Caution: For continued protection against risk of fire, replace only with same type and rating of fuse.

Product Disposal
This product contains electrical and electronic equipment.

1. As with other electrical equipment, the internal components of this product may contain hazardous materials. To avoid potential contamination of the environment, it is important that you dispose of this product according to local or regional waste administration systems and regulations.
2. Do not dispose of as unsorted municipal waste.
3. The crossed-out wheeled bin symbol shown indicates separate collection for electrical and electronic equipment.
4. You may contact us for further information regarding the environmental performance of this product.

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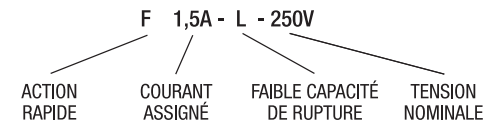
Distributed in USA by:
Evenflo Company, Inc.
1801 Commerce Drive
Piqua, Ohio 45356
1.866.99.AMEDA (26332)
www.ameda.com

Distributed in Canada by:
Distribué au Canada par:
Mothers Choice Products Ltd.
2133-11871 Horseshoe Way
Richmond, BC V7A 5H5
1.800.604.6225
www.motherschoiceproducts.com

Distributed in Australia by:
Advantage Health Care
34 Percy Street
Mordialloc VIC 3195
61 (3) 9580 9288
www.ameda.com.au

Distributed in UK by:
Ardo medical Ltd.
Unit 1, Belvedere Trading Estate
GB-Taunton, Somerset TA1 1BH UK
+44 (0) 1823 336 362
www.ardomedical.co.uk

Distributed in Switzerland
and Germany by:
Ardo medical AG
Gewerbstrasse 19
6314 Unterägeri, Switzerland
+41 (0) 41 754 70 70
www.ardo.ch

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