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Silicone Nipple Shield 
 Instructions

17212

For more information about Ameda Breastfeeding products, visit www.ameda.com.

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
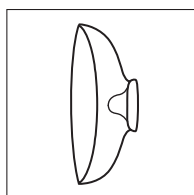
Distributed in USA by:
Evenflo Company, Inc.
 1801 Commerce Drive
 Piqua, Ohio 45356
1.866.99.AMEDA (263332)
www.ameda.com

Distributed in Canada by:
 Distribu  au Canada par:
Mothers Choice Products Ltd.
 2133-11871 Horseshoe Way
 Richmond, BC V7A 5H5
1.800.604.6225
www.motherschoiceproducts.com


Distributed in Australia by:
Advantage Health Care
 34 Percy Street
 Mordialloc VIC 3195
61 (3) 9580 9288
www.ameda.com.au

1

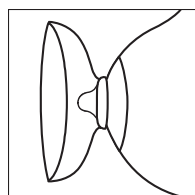2

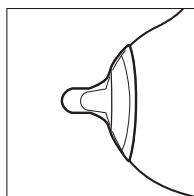3

Ameda Silicone Nipple Shield Instructions

The Ameda Nipple Shield may be used to help babies who are having problems latching to the breast. The Nipple Shield should be large enough to fit mother's nipple. It should also fit well into baby's mouth without gagging. If it is not a good fit for mother or baby, it should not be used.

Note: A nipple shield should only be used under the guidance of a healthcare professional, so that adequate milk intake can be ensured.

Before Each Use

Wash hands well with soap and water.

Before First Use

Wash in hot, soapy water before first use. Boiling is not necessary unless otherwise instructed by a healthcare provider. Rinse well in hot, clean water. Allow to fully air dry before use.

Using Ameda Nipple Shield

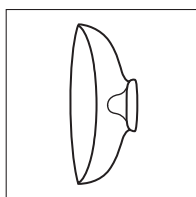
1. Bend back rim of Nipple Shield and about half of tip.
2. Place tip of Nipple Shield onto nipple.
3. As Nipple Shield is unfurled onto breast, nipple may be drawn into shield tip.

When latching baby to breast, be sure baby opens wide and latches well onto brim of shield, not just tip.


After breastfeeding, milk should be seen in tip of shield.

After Each Use

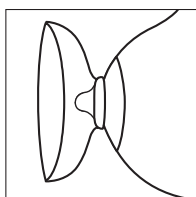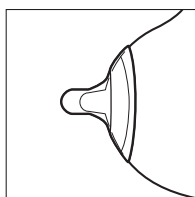
- Rinse off any milk with cool water.
- Wash in hot, soapy water.
- Rinse well in hot, clean water.
- Fully air dry.

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