
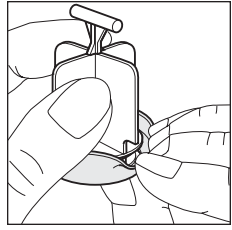
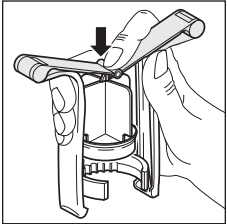
1


2

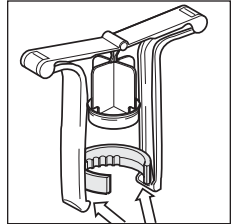
3

4

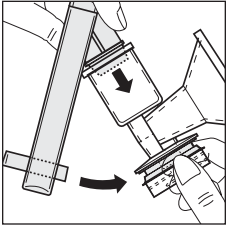5

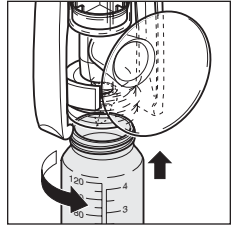6

EN One-Hand Breast Pump with Flexishield

Intended Use

The One-Hand Manual Breast Pump is intended to express and collect milk from the mother's breast to alleviate engorgement of the breast, maintain the ability of lactation, and provide mother's milk for future feedings when separation of mother and baby occurs.

Before First Use

Always wash hands well with soap and water before handling the products and pumping.

Cleaning

Unless told otherwise by a healthcare provider, follow these cleaning instructions.

Boil parts in water for 20 minutes before the first use and then once per day.

Assembly

1. Push the valve up firmly into lower portion of the breast flange.
2. Replace the seal over the bottom of the piston.
3. Press the spring down gently, place top of piston in the groove in the middle of the spring handle and release the spring.
4. Check that the snap ring is in place on both sides.
5. Push the piston into the upper chamber of the breast flange. Pull handle down and snap handle collar around bottom of flange (see shaded area in diagram).
6. Screw the milk bottle and breast flange together and tighten moderately.

Please take note of the enclosed instructions on the use of the Flexishield.

Pumping

Relax. Use both fast and slow squeezes. Watch your milk flow as you pump. Use it as your guide.

- a. Use a few minutes of fast, light squeezes to trigger milk flow.
- b. When milk flows, use slower, stronger squeezes. (Never so hard that it hurts.)
- c. For easier pumping, while milk is flowing "squeeze and hold".
- d. When milk flow slows, use faster squeezes again.
- e. When milk flows again, use slower, stronger squeezes.
- f. Repeat fast and then slow squeezes to pump more milk faster.
- g. Change breasts every 5-7 minutes for a total of 20 to 30 minutes.

Troubleshooting

Pumping with fast and slow squeezes is like a baby at the breast. Baby starts fast, then slow, and repeats. This two-speed action may help you pump more milk faster.

Is the pump not sucking strongly?

Disassemble the pump. Check the seal and valve for residues and cracks. Assemble the pump, making sure that the seal is attached correctly.

Daily Cleaning

See instructions on Cleaning Before First Use.

- After each use, take apart the One-Hand Breast Pump to clean.
- Rinse any milk from pump parts with cool water.
- Wash all parts in hot, soapy water.
- Rinse well with hot, clean water.
- Fully air dry parts on clean towel or drying rack before assembly and next use.

All parts are suitable for dishwashers, but are not microwaveable.

Clinics

121°C / 20 min 134°C / 5 min

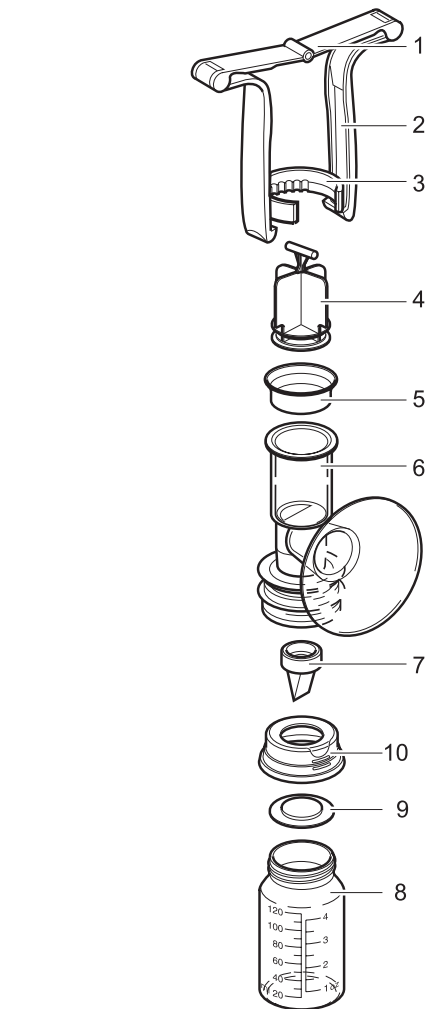
Notes

Ageing of the material is greatly accelerated by frequent sterilization. If breast flanges or milk bottles show discoloration or damage, discard them. Keep small parts and plastic bags out of the reach of children. The parts are not suitable for microwave ovens.

The milk bottle can be closed tightly with the locking rings and caps. Direct pumping into the Ameda Milk Freezer bags is also possible.

Disposal

The parts must be disposed of according to local regulations.

- | | |
|---|---|
| <p>EN</p> <ol style="list-style-type: none"> 1. Spring (POM) 2. Pump handle (POM) 3. Snap ring (POM) 4. Piston (PS, novolen) 5. Seal (silicone) | <ol style="list-style-type: none"> 6. Breast flange (PP) 7. Valve (silicone) 8. Milk bottle (PP) 9. Locking Ring (POM) 10. Locking cap (POM) |
| <p>DE</p> <ol style="list-style-type: none"> 1. Feder (POM) 2. Handgriff (POM) 3. Schnappring (POM) 4. Kolben (PS, novolen) 5. Dichtung (Silikon) | <ol style="list-style-type: none"> 6. Brustglocke (PP) 7. Ventili (Silikon) 8. Milchflasche (PP) 9. Verriegelungsring (POM) 10. Verriegelungskappe (POM) |

- | | |
|--|---|
| <p>FR</p> <ol style="list-style-type: none"> 1. Ressort (POM) 2. Poignée (POM) 3. Anneau de fixation (POM) 4. Piston (PS novolène) 5. Joint (silicone) | <ol style="list-style-type: none"> 6. Manchon de sein (PP) 7. Valve (silicone) 8. Flacon à lait (PP) 9. Anneau de verrouillage (POM) 10. Bouchon de verrouillage (POM) |
| <p>IT</p> <ol style="list-style-type: none"> 1. Molla (POM) 2. Impugnatura (POM) 3. Anello a scatto (POM) 4. Stantuffo (PS, novolen) 5. Guarnizione (silicone) | <ol style="list-style-type: none"> 6. Campana per il Seno (PP) 7. Valvola (silicone) 8. Bottiglietta del latte (PP) 9. Ghiera (POM) 10. Tappo di Chiusura (POM) |

- | | |
|---|---|
| <p>ES</p> <ol style="list-style-type: none"> 1. Resorte (POM) 2. Asa (POM) 3. Anillo de bloqueo (POM) 4. Émbolo (PS novolén) 5. Junta (silicona) | <ol style="list-style-type: none"> 6. Embudo de lactancia (PP) 7. Válvula (silicona) 8. Frasco para la leche (PP) 9. Anillo de cierre (POM) 10. Tapa de cierre (POM) |
| <p>PT</p> <ol style="list-style-type: none"> 1. Mola (POM) 2. Punho (POM) 3. Anel de bloqueio (POM) 4. Émbolo (PS, novolen) 5. Vedação (silicone) | <ol style="list-style-type: none"> 6. Tubo de sucção (PP) 7. Válvula (silicone) 8. Biberão para leite (PP) 9. Anel de bloqueio (POM) 10. Tampa de fecho (POM) |

- | | |
|--|---|
| <p>NL</p> <ol style="list-style-type: none"> 1. Veer (POM) 2. Handvat (POM) 3. Borgring (POM) 4. Zuiger (PS, novolen) 5. Afdichting (silicone) | <ol style="list-style-type: none"> 6. Kolfstuk (PP) 7. Ventiel (silicone) 8. Melkfles (PP) 9. Afsluitring (POM) 10. Afsluitkapje (POM) |
| <p>DK</p> <ol style="list-style-type: none"> 1. Fjeder (POM) 2. Håndtag (POM) 3. Springring (POM) 4. Stempel (PS, novolen) 5. Tætning (silikone) | <ol style="list-style-type: none"> 6. Brystflange (PP) 7. Ventili (silikone) 8. Mælkeflaske (PP) 9. Låsering (POM) 10. Låsehætte (POM) |

- | | |
|---|---|
| <p>NO</p> <ol style="list-style-type: none"> 1. Fjær (POM) 2. Håndgrepp (POM) 3. Snappring (POM) 4. Stempel (PS, novolen) 5. Pakkning (silikon) | <ol style="list-style-type: none"> 6. Brystkrage (PP) 7. Ventili (silikon) 8. Melkeflaske (PP) 9. Låsering (POM) 10. Låsehette (POM) |
| <p>SE</p> <ol style="list-style-type: none"> 1. Fjäder (POM) 2. Handgrepp (POM) 3. Snappring (POM) 4. Kolv (PS, novolen) 5. Packning (silikon) | <ol style="list-style-type: none"> 6. Bröstkupa (PP) 7. Ventili (silikon) 8. Mjölkflaska (PP) 9. Låsring (POM) 10. Låslock (POM) |

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