
Purely Yours Ultra

BREAST PUMP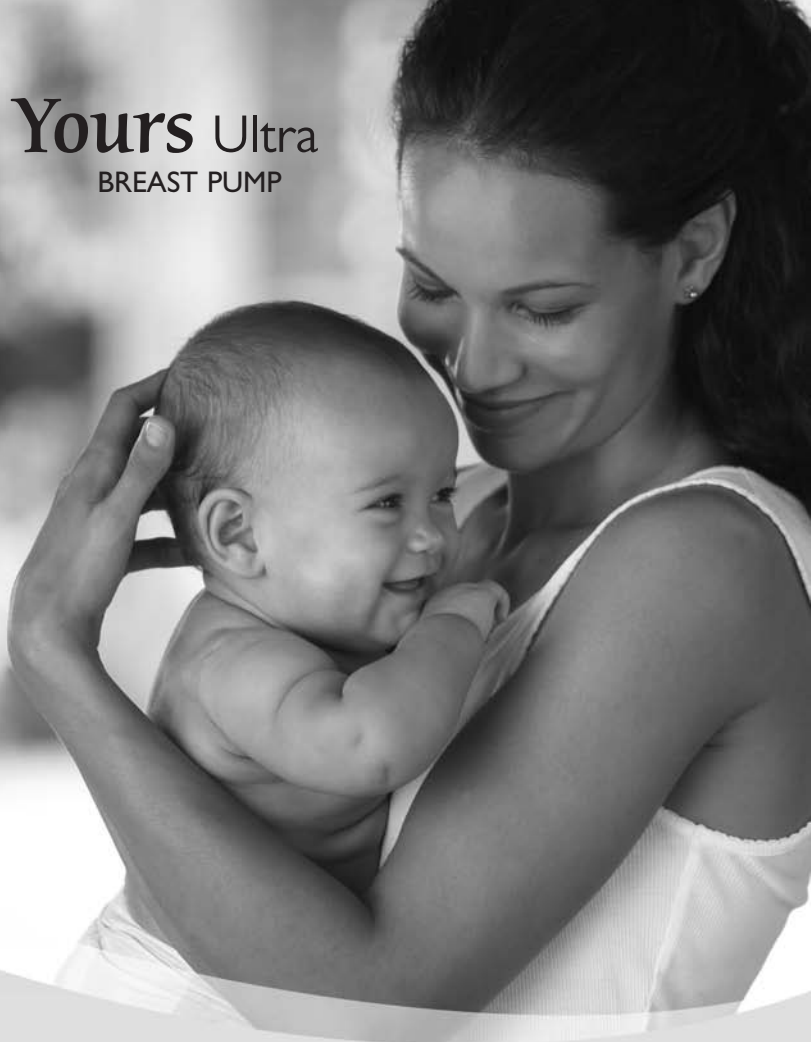

Purely Yours Ultra Breast Pump Instructions

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Contents

Important Safeguards	2	Troubleshooting the Purely Yours Ultra Breast Pump	9
Intended Use	2	Troubleshooting the HygieniKit System	9
Before Each Use	2	Adapting to the One-Hand Manual Breast Pump	10
Purely Yours Ultra Diagram	3	Set Up and Assembly	10
Purely Yours Ultra Breast Pump	4	<i>Parts Diagram</i>	10
Set Up and Assembly	4	<i>Converting from Purely Yours Ultra</i>	10
<i>Pump</i>	4	<i>Taking Pump Apart</i>	11
<i>Power Options</i>	4	Cleaning	12
HygieniKit Milk Collection System with CustomFit Flange System™	4	<i>Before First Use</i>	12
<i>Parts Diagram</i>	4	<i>After Each Use</i>	12
<i>Before First Use</i>	5	Using the One-Hand Breast Pump	12
<i>Assembly</i>	5	<i>Putting Pump to Breast</i>	12
<i>Converting from One-Hand Breast Pump</i>	5	<i>Checking Flange Fit</i>	13
<i>Attaching to Purely Yours Ultra Breast Pump</i>	6	<i>Pumping Made Simple</i>	13
<i>Single and Dual Pumping</i>	6	Troubleshooting the One-Hand Breast Pump	13
<i>Putting HygieniKit System to Breast(s)</i>	6	Milk Storage and Handling	14
<i>Checking Flange Fit</i>	6	<i>Containers</i>	14
Pump Controls	7	<i>Storage Guidelines</i>	14
<i>Using Control Dials</i>	7	<i>Thawing/Warming Milk</i>	14
<i>Starting Pump</i>	7	Flexishield Areola Stimulator	14
<i>Setting Suction</i>	7	Replacement Parts	15
<i>Setting Speed</i>	7	Technical Data	16
<i>Turning Pump Off</i>	7	Product Disposal	16
Pumping Made Simple	7	Warranty	17
<i>Milk Release or ‘Let-Down’</i>	7		
<i>Using Your Senses</i>	8		
<i>Along with Pump Controls</i>	8		
<i>Pumping Length</i>	8		
<i>Removing HygieniKit System from Breast(s)</i>	8		
Cleaning	8		
<i>Before Each Use</i>	8		
<i>After Each Use</i>	8		

IMPORTANT SAFEGUARDS

When using electrical products, especially if children are present, the following basic safety precautions should always be maintained.

READ ALL INSTRUCTIONS BEFORE USING

⚠ Danger: To reduce the risk of electrocution:

- Always unplug electrical devices immediately after use.
- Do not use while bathing, showering or swimming.
- Do not place or store where product can fall or be pulled into bathtub, sink or pool.
- Do not place or drop product into water or other liquid.
- Do not reach for electrical product that has fallen into water. Unplug from wall outlet immediately.

⚠ Warning: To reduce the risk of burns, electrocution, fire, or injury to persons:

- Never leave product unattended when plugged into electrical outlet.
- Close supervision is necessary when product used near children or invalids.
- Use product only for intended use as described in this manual.
- Do not use attachments other than those recommended by manufacturer.
- Never operate if product has damaged cord or plug, is not working properly, or has been dropped, damaged, or becomes wet.
- Operate Purely Yours Ultra Breast Pump from batteries or car adapter if reliability of electrical outlet is in doubt.
- Keep cord and all attachments away from heated surfaces.
- Never use while sleeping or drowsy.
- Do not use outdoors with cord.
- Do not operate where aerosol spray products are being used or oxygen is being administered.
- Batteries should be removed from pump when using the AC or Car adapter.

SAVE THESE INSTRUCTIONS

For Your Health and Safety

The Ameda Purely Yours Ultra Breast Pump is a personal care item and should never be shared between mothers or resold from one mother to another.

Intended Use


The Ameda Purely Yours Ultra Breast Pump is intended to express and collect milk from the mother's breast, to alleviate engorgement of the breast, maintain the ability of lactation, and provide mother's milk for future feedings when separation of mother and baby occurs.

Before Each Use

Always wash hands well with soap and water before handling the Purely Yours Ultra, One-Hand Breast Pump, HygieniKit Milk Collection System, or other Ameda products. Do not use solvents or abrasives.

Purely Yours Ultra Breast Pump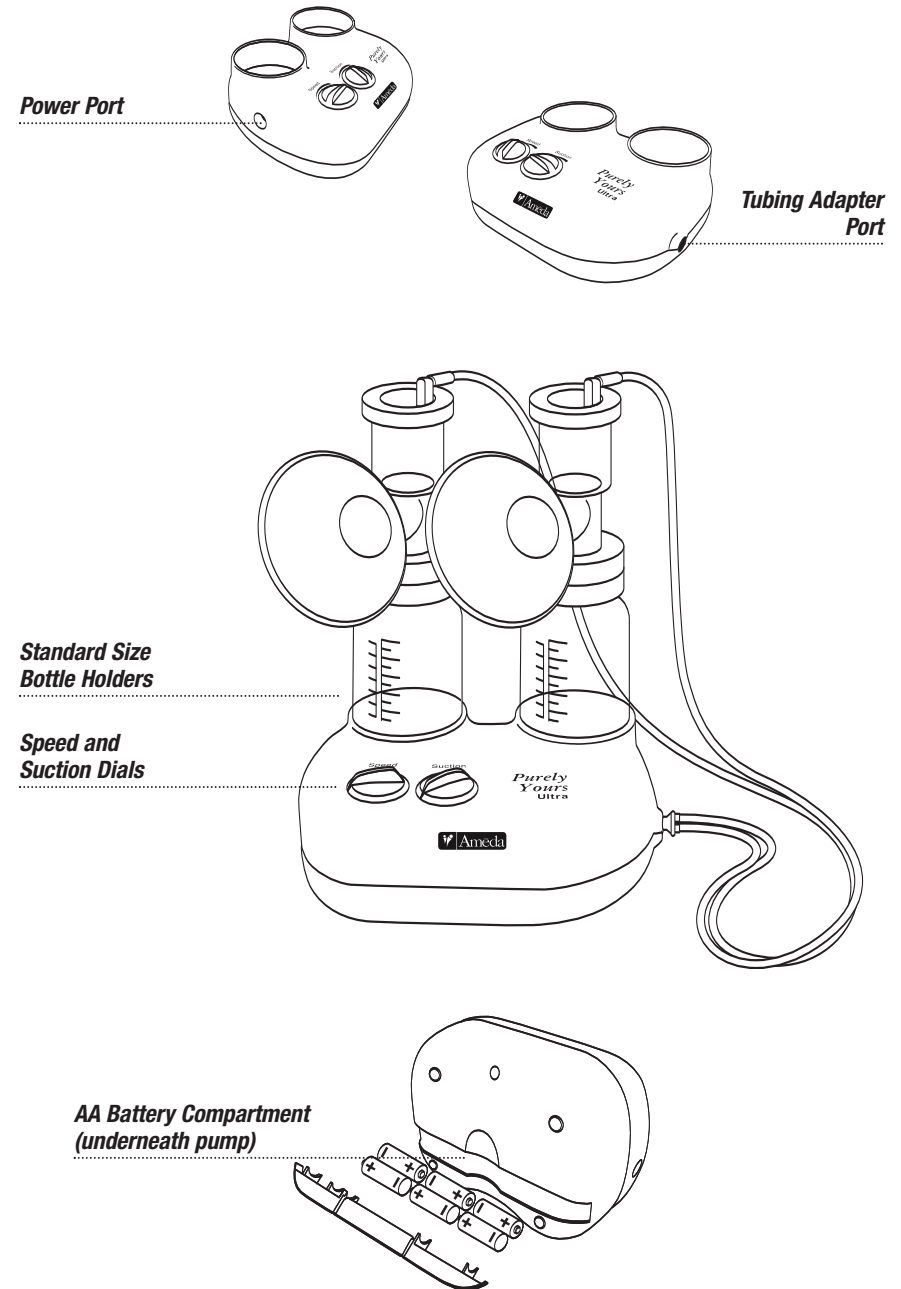

Purely Yours Ultra Breast Pump


Set Up and Assembly

Pump

Remove pump motor from box or case. Choose power option.

Power Options


The Purely Yours Ultra Breast Pump has three power options: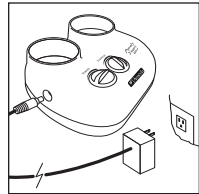

AC Adapter (included)

Use only the AC adapter supplied with the Purely Yours Ultra. If condition of AC adapter is in doubt, contact an Ameda retailer to replace.


- Connect AC adapter to power port on left side of pump.
- Plug AC adapter into electrical outlet.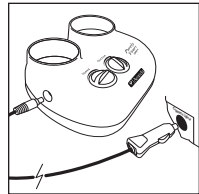

Car Adapter (included)

Use only with the car adapter specified for the Purely Yours Ultra (see "Technical Data").

- Connect car adapter to power port on left side of pump.
- Insert other end into lighter input of vehicle.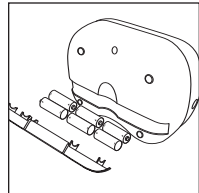

6 AA Alkaline or Rechargeable Batteries (not included)


- Open battery cover on underside of pump.
- Insert 6 AA batteries. Make sure (+) and (-) battery ends are facing the right way.
- Match up triangle points on the battery compartment and base of breast pump. Gently snap into place.

Remove batteries if pump will not be in use for a long time. Dispose of batteries properly.

When batteries are inside pump and AC adapter or car adapter is plugged in, pump will not draw power from batteries.

Warning: Batteries should be removed from pump when using the AC or Car adapter.

HygieniKit Milk Collection System with CustomFit Flange System™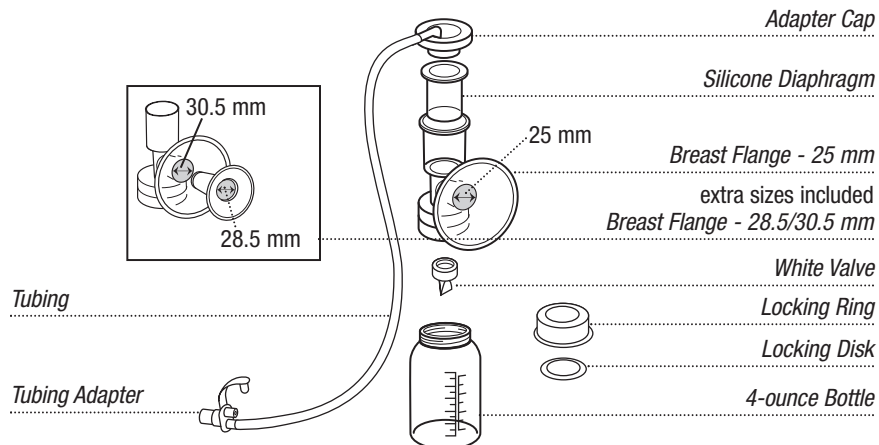

ALWAYS WASH HANDS WELL WITH SOAP AND WATER BEFORE PUMPING.

Before First Use


If your HygieniKit system is marked "STERILE," open and use.

If your HygieniKit system is **NOT** marked "STERILE," before first use:

- Boil all parts—except tubing, adapter cap, and tubing adapter—for 20 minutes.
- Remove all parts from water right after boiling.
- Fully air dry before assembly and use.

Do not boil or wash tubing, adapter cap, and tubing adapter.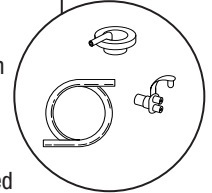


NOTE: Before using the HygieniKit system, check the parts for cracks, chips, tears, color changes, or breakdown. They can affect the function of the kit. If parts need replacing due to defects, please call:

In US: 1.866.99.AMEDA (26332)

In Canada: 1.800.604.6225


In other countries: Call your local distributor or location you purchased pump. For listing of distributors in your country, please visit www.ameda.com.

Assembly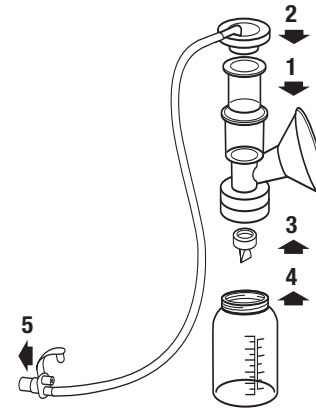


1. Insert diaphragm into upper chamber of breast flange.
2. Snap adapter cap with tubing onto the top of upper chamber.
3. Gently push valve firmly onto ring on underside of breast flange.
4. Screw bottle onto suitable size breast flange.
5. Push tubing adapter at end of tubing into tubing adapter port on pump.


Converting from One-Hand Breast Pump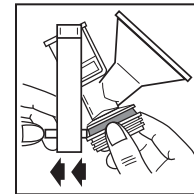

1. Detach handle collar from breast flange and lift piston out of upper chamber.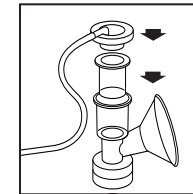

2. Place diaphragm into upper chamber.
3. Snap adapter cap with tubing onto upper chamber.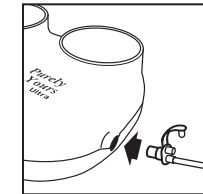

4. Insert tubing adapter into tubing adapter port on the Purely Yours Ultra electric breast pump.


Attaching to Purely Yours Ultra Breast Pump

ALWAYS WASH HANDS WELL WITH SOAP AND WATER BEFORE PUMPING.

The HygieniKit system may be used to single or dual pump with the Purely Yours Ultra Breast Pump. Simply insert the tubing adapter into the tubing adapter port of the pump.


Single and Dual Pumping

The HygieniKit system can be used to single or dual pump.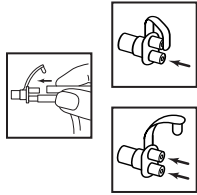

To single pump:
Remove one tube and close tubing adapter.

To dual pump:
Both tubes should be attached to tubing adapter.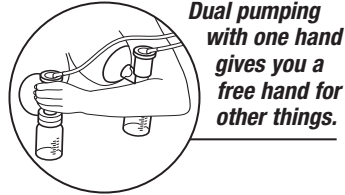

Dual pumping with one hand gives you a free hand for other things.

Putting HygieniKit System to Breast(s)

To begin pumping:


- Center nipple(s) in breast flange(s).
- Fill breast flange(s) with the breast(s) to create an air seal.
- Angle breast flange(s) slightly downward so milk flows into bottle(s).

Checking Flange Fit

Some Ameda HygieniKits include one size breast flange, with an opening of 25mm. Other Ameda kits include three flanges sizes (25, 28.5, and 30.5mm). Many mothers do well with the 25mm standard size breast flange, but you may need a larger flange (30.5mm) or insert (28.5mm) if pumping hurts, even on low suction. If the standard flange is too tight, a larger flange or insert may feel better and pump more milk. Flange fit can change as the breast changes after birth and with pumping. Use the size that feels best and gives the best results. If the 30.5mm size is not large enough, contact your Ameda retailer to obtain the 32.5/36 mm Ameda Custom Breast Flanges.


Visit www.ameda.com for more information.

To check your flange fit, watch your nipple during pumping.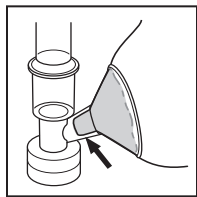

Good Fit

If you see space around your nipple, you have a good fit.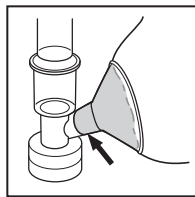

Tight Fit


If your nipple rubs against the sides of the flange, it is too tight.

To use the larger flanges:

- Replace standard flange with larger flange
- Move valve to larger flange
- See instructions to assemble other parts
- Try the larger flange with and without the insert on each breast until you find the size that feels better and/or pumps more milk.

Pump Controls

Using Control Dials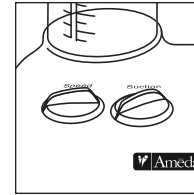

Turn dials to the right (clockwise) to increase pump settings (faster SPEED, stronger SUCTION). Turn dials to the left (counter-clockwise) to decrease pump settings (slower SPEED, gentler SUCTION).

Starting Pump

- Choose power option, and connect pump to power source.
- Turn SUCTION dial to the right until light turns on. Pump is now on.

Setting Suction

- Pump starts at lowest SUCTION setting.
- Center nipple(s) in breast flange(s) and fill breast flange(s) with breast(s) to create an air seal.
- Increase suction by turning SUCTION dial to the right (clockwise) until you feel slight discomfort.
- Decrease SUCTION slightly until pumping feels good.
- Begin pumping at the highest comfortable SUCTION setting.

Setting Speed

Many mothers find that fast speed triggers milk flow and slow speed drains it faster. Watch your milk flow as you pump. Use it as your guide.

- Start with SPEED dial all the way to the right (clockwise) at the fastest speed.
- Once milk is flowing (let-down), turn SPEED dial to the left (counter-clockwise) to a slow setting. This may help drain breasts faster.
- Once milk is flowing, readjust SUCTION to highest comfortable setting.
- When milk flow slows to a trickle, turn SPEED dial to the right (clockwise) to the fastest speed to trigger another let-down.
- At next let-down, turn SPEED dial back to the left (counter-clockwise) back to a slow setting.
- Repeat steps as needed to pump more milk faster.

Turning Pump Off

- When done pumping, turn SUCTION dial all the way to the left (counter-clockwise) to turn pump off.

Pumping Made Simple

Does a pump need to be set at the strongest suction to get more milk? No. Set your pump at the highest suction that feels good...and no higher.

Milk Release or 'Let-Down'

Pumping milk is not like sucking liquid through a straw. With a straw, the stronger you suck, the more liquid you get. When pumping, the milk comes only when a let-down, or milk release, happens. Without a milk release, most milk stays in the breast.

What is a milk release?

- Hormones cause muscles in the breast to squeeze and milk ducts to widen, pushing the milk out.
- Some mothers feel tingling. Others feel nothing.
- A milk release can happen with a touch at the breast, hearing a baby cry, or even by thinking about your baby. Feeling your baby's soft, warm skin against yours can cause a milk release.
- Feelings of stress, anger, or upset can block milk release.

Using Your Senses

While breastfeeding, most mothers have three or four milk releases without knowing it. To get more milk with your pump, you need more milk releases. But you may need some help at first until your body learns to respond to your pump like a baby.

To trigger more milk releases, use your senses. One or two senses may work better than the others, so try them all to find out which work best for you.

- **Mind/Feelings:** Close your eyes, relax, and imagine your baby breastfeeding. Think about how much you love your baby.
- **Sight:** Look at your baby or your baby's photo.
- **Hearing:** Listen to a tape of your baby cooing or crying. If you're away, call and check on your baby. Or call someone you love to relax and distract you.
- **Smell:** Smell your baby's blanket or clothing.
- **Touch:** Apply warm cloths or gently massage your breasts.
- **Taste:** Sip a favorite, warm non-alcoholic drink to relax you.

Along with Pump Controls

To get more milk faster, use your senses along with your pump controls.

- Each time you turn your SPEED dial to fast, use those senses that work for you.
- This may help trigger another milk release faster.
- The less time spent waiting for a milk release, the more milk you get while pumping.

Pumping Length

Talk to your healthcare provider about how long you should pump. In most cases, 10 to 15 minutes per breast is suggested.

- If single pumping, 20 to 30 minutes total.
- If dual pumping, 10 to 15 minutes total.

Removing the HygieniKit System from Breast(s)

For greater comfort, after turning pump off:

- Insert a finger into breast flange(s) to break suction.
- Remove HygieniKit system from breast(s).


Cleaning

Before Each Use



Always wash hands well with soap and water before handling the One-Hand Breast Pump, HygieniKit milk collection system, or other Ameda products. Do not use solvents or abrasives.

After Each Use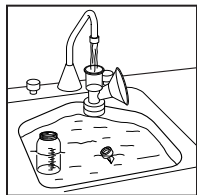

- Take apart HygieniKit system to clean.
- Rinse any milk from pump parts with cool water.
- Wash all parts—except tubing, adapter cap, and tubing adapter—in hot, soapy water.
- Rinse well with hot, clean water.

Do not boil or wash tubing, adapter cap, and tubing adapter.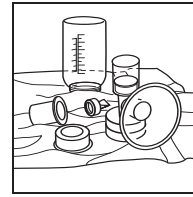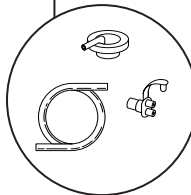

- Fully air dry parts on clean towel or drying rack before assembly and next use.
- Cover parts with clean towel if not planning to use again soon.

Breast flanges and bottles can be washed on the upper rack of a dishwasher. Wash diaphragms and white valves gently by hand. **Do not insert anything into white valve while cleaning.**

Troubleshooting the Purely Yours Ultra Breast Pump

To check pump suction from motor:

- Remove tubing adapter from tubing adapter port of pump.
- Put a finger over tubing adapter port.
- If suction is felt, pump motor is working fine.
- If suction is not felt, unplug AC adapter, wait a few seconds, plug back into electrical outlet.

The Purely Yours Ultra has a reset feature. If the Purely Yours Ultra motor stops running:

- Unplug AC adapter cord from electrical outlet.
- Wait a few seconds and then plug AC adapter back into outlet.

Troubleshooting the HygieniKit Milk Collection System

If milk is dripping too slowly into bottle:

- Set SPEED dial to a faster setting.
- Replace silicone diaphragm(s) and white valve(s).

If pumping hurts:

- Set pump to lower SUCTION setting.
- Check pump fit. Larger flanges may be needed.

If suction is low:

- Check that white valve(s) is/are pushed well onto breast flange(s).
- Check white valve(s) for tears or stretches. Replace if needed.
- If single pumping, check tubing adapter to be sure side not in use is closed.
- Check tubing adapter for cracks. If seen, replace.
- Set pump to higher SUCTION setting.


The white valves are vital for proper suction. To work well, they must be intact and look closed when pump is not in use. **To prolong white valve life:**

- When removing from breast flange, pull gently from thick sides, not thin edge.
- Do not stick anything into valves during cleaning. Just swish them around in hot, soapy water, rinse well in hot water, and air dry.


Adapting to the Ameda One-Hand Manual Breast Pump

The One-Hand Handle Assembly is included with the Purely Yours Ultra. It can also be bought separately to convert the HygieniKit system to the One-Hand Manual Breast Pump.


Set Up and Assembly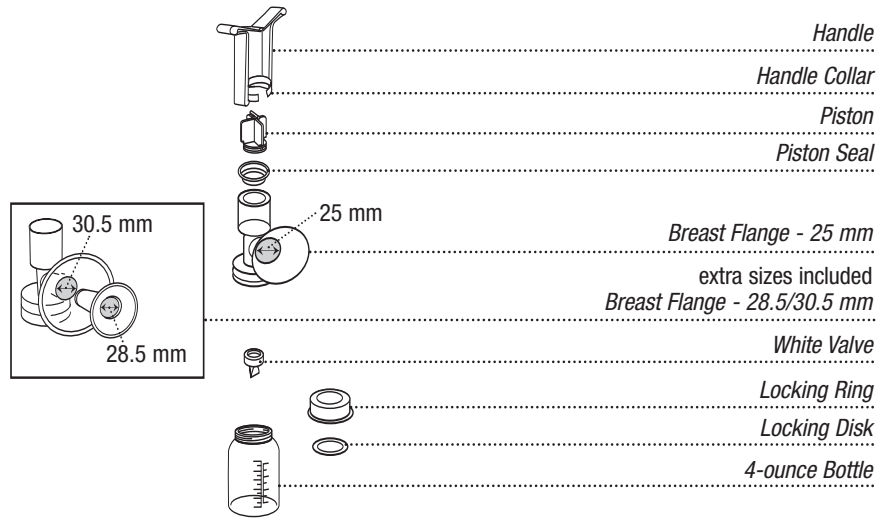


Converting from Purely Yours Ultra

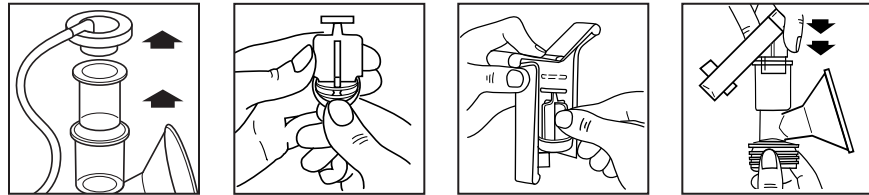
1. Unsnap and remove adapter cap.
2. Remove diaphragm from upper chamber of breast flange.
3. Push piston seal onto base of piston.
4. Remove plastic covering on the One-Hand handle assembly. Push center of handle down and snap top of piston into groove on handle.
5. Push piston into upper chamber of flange.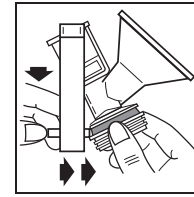

6. Pull handle down and snap handle collar around groove of breast flange.
(See shaded area in diagram above.)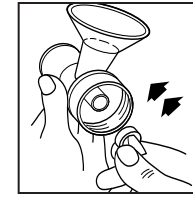


7. Gently push valve firmly onto ring on underside of breast flange.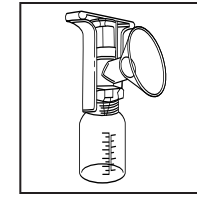


8. Screw flange onto bottle.
Handle is on correctly if top of handle is a straight line.

Taking Pump Apart


Unscrew Bottle from Breast Flange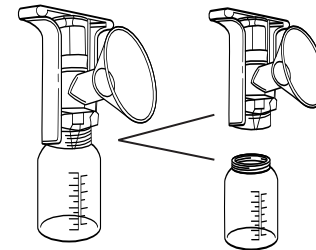


Slide Handle Bottom Off Breast Flange

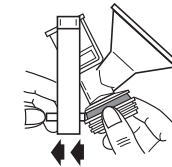
Remove Piston from Upper Chamber

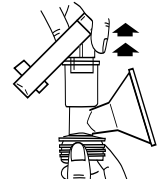
Push Down on Handle to Release Piston

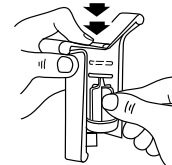Peel Off Seal from Piston

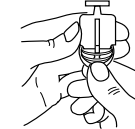Pull Off Valve from Breast Flange

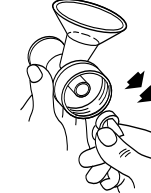Note: Grasp valve gently by thick sides, not by thin edge

Cleaning

Always wash hands well with soap and water before handling the One-Hand Breast Pump, HygieniKit milk collection system, and other Ameda products. Do not use solvents or abrasives.

Before First Use


If your One-Hand Breast Pump is marked “STERILE,” open and use.


If your One-Hand Breast Pump is **NOT** marked “STERILE,” before first use:

- Boil all parts for 20 minutes.
- Remove all parts from water right after boiling.
- Fully air dry before assembly and use.

After Each Use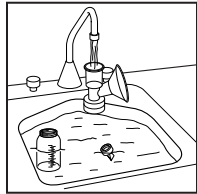

- Take apart One-Hand Breast Pump.
- Rinse any milk from pump parts with cool water.
- Wash all parts in hot, soapy water.
- Rinse well in hot, clean water.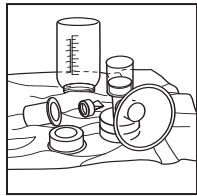

- Fully air dry parts on clean towel or drying rack before assembly and next use.
- Cover parts with clean towel if not planning to use again in two to three hours.


Breast flanges, handle, and bottles can be washed on the upper rack of a dishwasher. Wash piston seal and white valve gently by hand.

Do not insert anything into white valve while cleaning.


Using the One-Hand Breast Pump

ALWAYS WASH HANDS WELL WITH SOAP AND WATER BEFORE PUMPING.


Putting Pump to Breast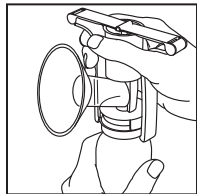


1. Place hand high on handle, thumb in front of upper chamber, fingers behind (If left-handed, reverse).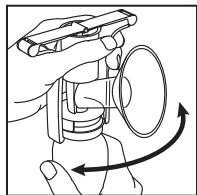


2. Swivel handle for wrist comfort.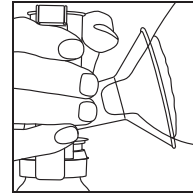

3. Center nipple in breast flange.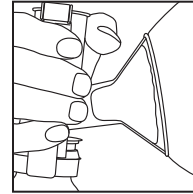

4. Hold flange to breast to make an air seal.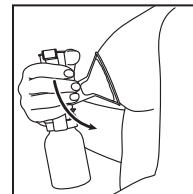

5. Tilt flange slightly downward so milk flows into bottle.

Checking Flange Fit (Refer to the *Checking Flange Fit* section on page 6)

Pumping Made Simple

Use both fast and slow squeezes. Watch your milk flow as you pump. Use it as your guide.

- Use a few minutes of fast, light squeezes to trigger milk flow.
- When milk flows, use slower, stronger squeezes. (Never so hard it hurts.)
- For easier pumping, while milk is flowing, “squeeze and hold.”
- When milk flow slows, use faster squeezes again.
- When milk flows again, use slower, stronger squeezes.
- Repeat fast then slow squeezes to pump more milk faster.
- Change breasts every 5-7 minutes for a total pumping time of 20-30 minutes.

Pumping with fast and slow squeezes is like a baby at the breast. Baby starts fast, then slow, and repeats. This two-speed action may help you pump more milk faster.

Troubleshooting the One-Hand Breast Pump

If milk is not dripping into bottle fast enough:

- And you are using “squeeze and hold,” release the handle.
- If not using “squeeze and hold,” release handle more often.


If pumping hurts:

- Use gentler handle squeezes.
- Check flange fit. A larger flange may be needed.

If suction is low:

- Check the white valve is on fully.
- Check valve for tears or stretches. Replace if needed.
- Remove piston seal, stretch out on flat surface, and replace on piston.

Peel Off Seal
from Piston

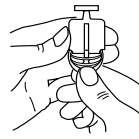Stretch Seal

Milk Storage and Handling

Containers

– ALWAYS DATE CONTAINERS –

Human milk should be stored in a dated, clean, sealed container. Glass or hard plastic containers or milk freezer bags can be used.

Guidelines for Storing and Thawing Human Milk

Storage Time for Human Milk*	Deep Freeze	Refrigerator Freezer	Refrigerator	Cooler with Frozen Ice Packs	Room Temperature	
	(0°F/-18°C)	(variable 0°F/-18°C)	(39°F/4°C)	(59°F/15°C)	(66°F–72°F) (19°C–22°C)	(72°F–79°F) (22°C–26°C)
Fresh	12 Months	3–4 Months	8 Days	24 Hours	6–10 Hours	4 Hours
Frozen, Thawed in Fridge	Do Not Refreeze	Do Not Refreeze	24 Hours	Do Not Store	4 Hours	4 Hours
Thawed, Warmed, Not Fed	Do Not Refreeze	Do Not Refreeze	4 Hours	Do Not Store	Until Feeding Ends	Until Feeding Ends
Warmed, Fed	Discard	Discard	Discard	Discard	Until Feeding Ends	Until Feeding Ends

* Storage times may vary for premature or sick babies. Sources: Jones, F. and M.R., Tully, *Best Practice for Expressing, Storing and Handling Human Milk in Hospitals, Homes and Child Care Settings*. Raleigh, NC: Human Milk Banking Association of North America, 2006; Mohrbacher, N. and J. Stock, *The Breastfeeding Answer Book*. Schaumburg, IL: La Leche League International, 2003.

Thawing/Warming Milk

Thaw frozen milk in refrigerator or thaw or warm milk by placing container in cool then warm water.

– Do Not Microwave Human Milk –









Microwaving changes the milk. It also creates hot spots in the milk that can burn the baby's mouth.

Flexishield Areola Stimulator (optional accessory)

The Flexishield Areola Stimulator:

- Is a flexible silicone breast flange insert
- Gives a soft feel against the skin while pumping
- Compresses the breast while pumping to help trigger milk release
- Narrows the breast flange, so works best with mothers with small nipples
- Fits only the standard breast flange (see “Checking Flange Fit”)

Replacement Parts and Accessories

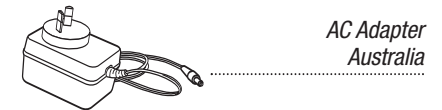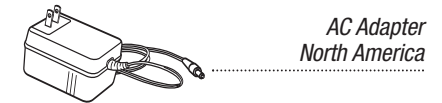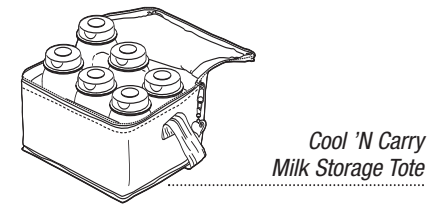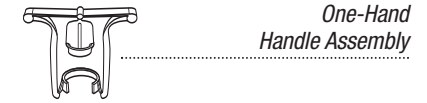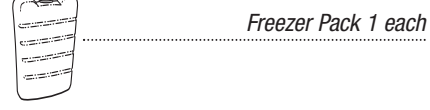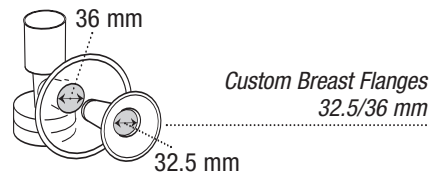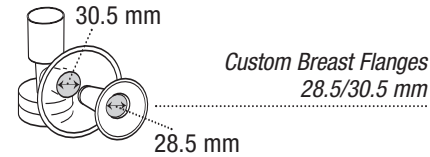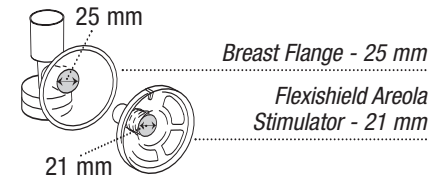Technical Data

Power Requirements

AC Adapter and Car Adapter *(included)*

AC Adapter – North American Plug Configuration

9VDC/1.3A, 120V/60Hz, Ameda Stock No. 622401, direct plug in AC power adapter.

AC Adapter – Australian Plug Configuration

9VDC/1.0A, 240V/50Hz, Ameda Stock No. 402707, direct plug in AC power adapter.

AC Adapter – United Kingdom Plug Configuration

9VDC/1.0A, 240V/50Hz, Ameda Stock No. 402705, direct plug in AC power adapter.

AC Adapter – Europe Plug Configuration

9VDC/1.0A, 240V/50Hz, Ameda Stock No. 402706, direct plug in AC power adapter.

Car Adapter

12VDC, F1.5A L250V fused adapter.

Batteries *(not included)*

Six AA batteries, 1.5 V alkaline or 1.2 V rechargeable

Regulatory Compliance

UL and CSA approved with North American power adapter only.

UL File Number E189700

CSA File Number LR76525

CE0086

Product Disposal

This product contains electrical and electronic equipment.

1. As with other electrical equipment, the internal components of this product may contain hazardous materials. To avoid potential of this product according to local or regional waste administration systems and regulations.
2. Do not dispose of as unsorted municipal waste.
3. You may contact us on 1.800.866.99.AMEDA (26332) for further information regarding the environmental performance of this product.

Warranty

Ameda Purely Yours Ultra Electric Breast Pump

Limited Consumer Warranty (United States and Canada)

The Ameda Purely Yours Ultra Electric Breast Pump (“Product”) is warranted to the original user (“Customer”) only. Use of the Product by more than one person will void this express limited warranty.

Evenflo Company, Inc. (“Evenflo”) warrants that it will convey to the Customer good title to the Product. From the date of original purchase of this Product, Evenflo warrants the Product to the Customer against defects in material or workmanship for one year on the pump mechanism and 90 days on all other pump components. Evenflo’s sole obligation under this limited warranty shall be to repair or replace, at Evenflo’s option, any Product that is determined to be defective by Evenflo and determined to be covered by this limited warranty. Repair or replacement under this limited warranty is the sole and exclusive remedy of the Customer. **Proof of purchase in the form of a receipted invoice or bill of sale evidencing that the Product is within the warranty period must be presented to obtain warranty service.** This limited warranty is extended by Evenflo ONLY to the Customer and is not assignable or transferable to subsequent purchaser or end users of the Product. For warranty service contact Ameda at 1.866.99.AMEDA (26332).

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE ON THE PRODUCT IS HEREBY DISCLAIMED. EVENFLO SHALL NOT BE LIABLE FOR ANY INCIDENTAL, CONSEQUENTIAL, SPECIAL OR PUNITIVE DAMAGES OF ANY KIND FOR BREACH OF THE EXPRESS LIMITED WARRANTY ON THE PRODUCT OR ANY WARRANTY IMPLIED BY OPERATION OF LAW. OTHER THAN THE EXPRESS LIMITED WARRANTY SET FORTH ABOVE, THERE ARE NO OTHER WARRANTIES THAT ACCOMPANY THIS PRODUCT AND ANY ORAL, WRITTEN OR ANY OTHER EXPRESS REPRESENTATION OF ANY KIND IS HEREBY DISCLAIMED.

Non-Applicability of Warranty

In addition to any other limitations on the warranty discussed above, the warranty shall expire and be of no force or effect upon the occurrence of any of the following:

- (i) the cause of damage to the Product is due to improper handling or use.
- (ii) the Product is not properly maintained by the user or the Customer as determined by Evenflo.
- (iii) non-Ameda approved accessories or spare parts are used with the Product as determined by Evenflo.

If any of the above occurs, Evenflo shall not be responsible for damage to the Product, parts thereof or injuries arising therefrom, either directly or indirectly. The warranty set forth above replaces all prior warranties with respect to the purchased Product, whether in writing or otherwise.

In Other Countries: Call your local distributor or location where you purchased Product. For listing of distributors in your country, please visit www.ameda.com.

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For more information, call 1.866.99.AMEDA (26332) or visit ameda.com for the complete line of Ameda breastfeeding products, articles and advice devoted to breastfeeding and pumping mothers.

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CE 0086

EC REP CanReg Inc.
Carlingford, County Louth, Ireland

Evenflo Company, Inc.
Piqua, Ohio, USA

Ameda, Flexishield, Purely Yours Ultra, HygieniKit, CustomFit Flange System and associated logo are trademarks of Evenflo Company, Inc.

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©2008 Evenflo Company, Inc.
Printed in the USA.
910239-0508

Distributed in USA by:
Evenflo Company, Inc.
1801 Commerce Drive
Piqua, Ohio 45356
1.866.99.AMEDA (26332)
www.ameda.com

Distributed in Canada by:
Distribué au Canada par :
Mothers Choice Products Ltd.
2133-11871 Horseshoe Way
Richmond, BC V7A 5H5
1.800.604.6225
www.motherschoiceproducts.com

Distributed in Australia by:
Advantage Health Care
34 Percy Street
Mordialloc VIC 3195
61 (3) 9580 9288
www.ameda.com.au