Ameda Pearl Electric Breast Pump

QUICK START CARD

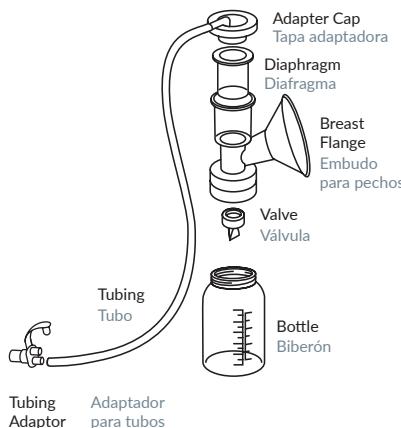
GETTING STARTED: Before first use, wash hands thoroughly with soap and water before touching breast pump, kit and breasts.

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Ameda
Trusted by Hospitals, Loved by Moms.

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1

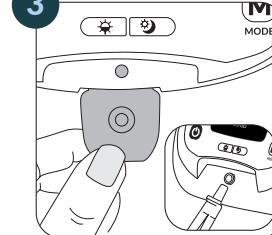
- If kit is sterile, use preassembled kit immediately.
- If not sterile, clean, sanitize and assemble the kit (follow cleaning instructions on the back of this card).

2

- First connect one end of the power adapter to the back of the pump, then the other end to an electrical outlet.

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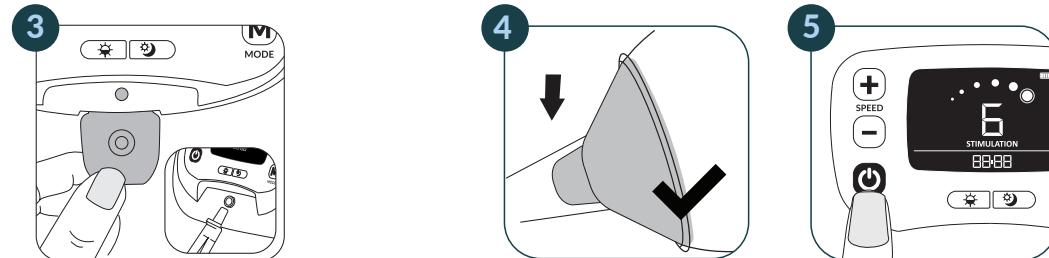
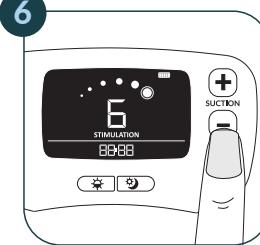
6


Adjust Stimulation Suction

- Adjust the Stimulation Suction setting gradually to your **preferred comfort** level (maximum 6).

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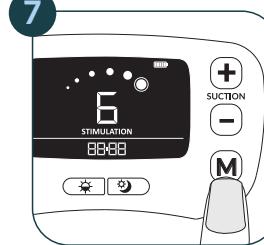7


Change Mode

- Once your milk starts to flow using the STIMULATION MODE, press the Mode (M) button to switch into Expression Mode.

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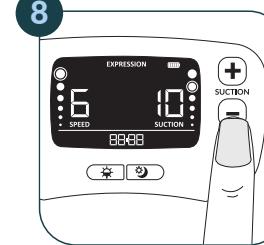8


Adjust Expression Suction Level

- Adjust the Expression Suction setting gradually to your **preferred comfort** level. (maximum 12).

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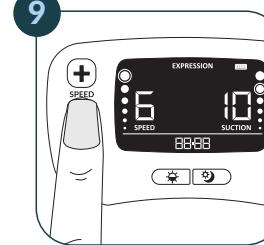
9

Adjust Speed

- Adjust the speed in Expression Mode to your **preferred comfort** level (maximum 6).
- Once your milk stops flowing freely while in Expression Mode, press the Mode button to go back to Stimulation Mode to induce a second let-down if desired.

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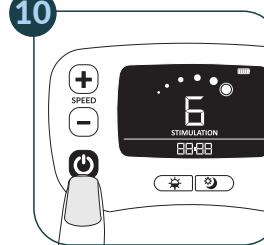End Pumping Session

- When finished pumping, press power button to turn off.

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Cleaning the Ameda Pearl Electric Breast Pump

For At-Home/End Users:

After each pumping, turn pump OFF. Wipe with clean, damp cloth.

For Hospital/Rental Stations:

Turn pump OFF. Clean between users wearing disposable gloves. Use a hospital approved antibacterial cleaner such as CaviCide®, CaviWipes®, or PDI® Sani-Cloth® AF3.

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
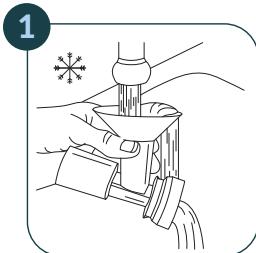
Cleaning the Ameda HygieniKit


IMPORTANT: Do Not Wash Tubing Adapter, Silicone Tubing, Adapter Cap. They do not contact milk during pumping.

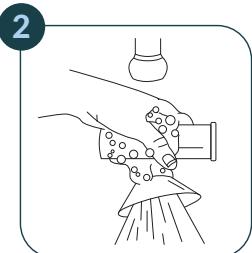
RINSE any milk from kit parts with cool water.

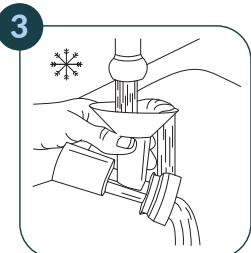Hand WASH parts with warm, soapy water. Do not insert anything into valve or diaphragm while cleaning.

RINSE well with warm, clean water.

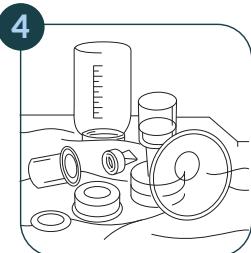Air-dry thoroughly on a clean surface.

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Breast Pump Valve and Diaphragm Check

Inspect valves and diaphragms for tears or holes. Replace any damaged valves and diaphragms prior to pumping. Replace valves every 8-12 weeks and diaphragms every 4-6 weeks.

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SCAN ME

Consult Instructions for Use
Ameda Pearl Electric Breast Pump
and HygieniKit

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FIND YOUR BEST FIT

A good breast flange fit is essential for comfort and pumping efficiency.

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Good Fit

During pumping, nipple moves freely in the breast flange tunnel. You see space around the nipple. Not much areola is drawn into the tunnel.

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Too Small

During pumping, nipple rubs against the sides of the tunnel.

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Too Large

During pumping, areola is drawn into the flange and may rub against the side of the tunnel.

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